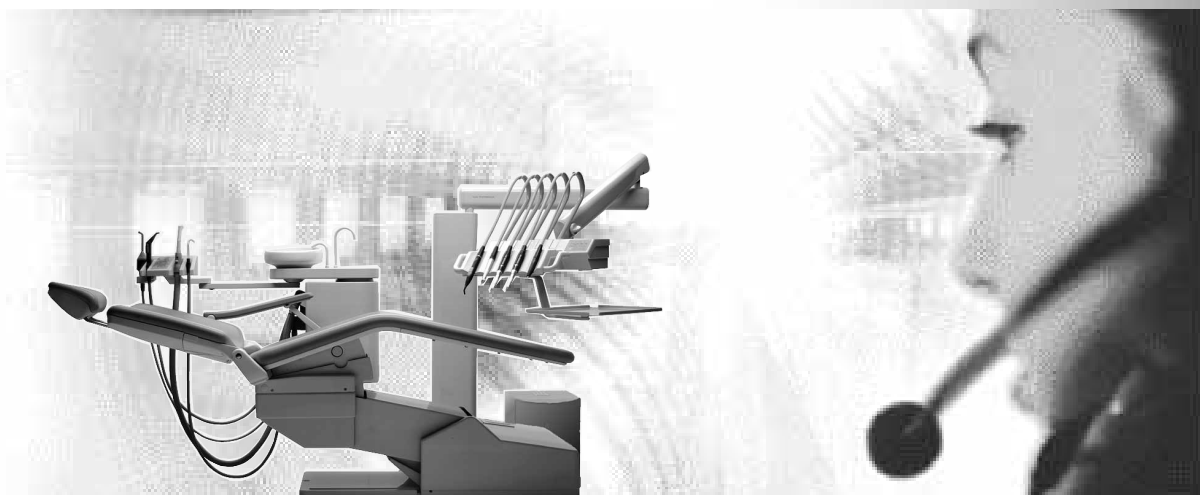
Documents I.
KaVo SYSTEMATICA® 1060.

Always on the safe side.

KaVo. Dental Excellence.

Vertrieb/distribution:

KaVo Dental GmbH & Co. KG

Bahnhofstraße 20 • D-88447 Warthausen

Tel.: 0 73 51 / 56-0 • Fax: 0 73 51 / 1 22 14

Hersteller/manufacturer:

Kaltenbach & Voigt GmbH & Co. KG

Bismarckring 39

D-88400 Biberach

A1	Benutzerhinweise Instructions de service	User notes Istruzioni per l'uso	A1
A2	Gerätebeschreibung Description de l'équipement	Unit description Descrizione degli apparecchi	A2
A3	Bedienung allgemein Généralités concernant les commandes	General operation Istruzioni per generali per l'uso	A3
A4	Bedienung Patientenstuhl Commande fauteuil	Operating the dental chair Comando della poltrona	A4
A5	Bedienung Fußanlasser Commande rhéostat	Operating the foot control Utilizzo del reostato a pedale	A5
A6	Bedienung Arztelement Commande ensemble praticien	Operating the dentist's unit Elemento dentista	A6
A7	Bedienung Helferinelement Commande ensemble assistante	Operating the assistant's unit Elemento assistente	A7
A8	Pflege Entretien	Maintenance and care Manutenzione	A8
A9	Beheben von Störungen Suppression des anomalies de fonctionnement	Troubleshooting Eliminazione di difetti	A9
A10	Adaptierbares Zubehör Accessoires adaptables	Optional accessories Accessori integrabili nel riunito	A10
A11	Zusatzanweisungen Instructions complémentaires	Additional instructions Istruzioni supplementari	A11

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A 1	User notes.....	6
A 1.1	Important notes	6
A 1.1.1	Safety checks	10
A 1.2	Equipment cleaning and maintenance	12
A 1.3	Equipment disinfection	14
A 1.4	Sterilization and disinfection of instruments	14
A 1.5	Improper use	16
A 1.6	Technical customer service	18
A 2	Unit description	20
A 2.1	KaVo SYSTEMATICA 1060.....	20
A 2.2	Dentist's unit	22
A 2.3	Assistant's unit	24
A 2.4	Dentist's / assistant's keypad	24
A 2.5	Location of rating plate	26
A 2.6	Explanation of rating plate	26
A 2.7	Location of nameplates	28
A 3	General operation	30
A 3.1	Turning the main switch on	30
A 3.2	Loading the demo program	30
A 3.3	Headrest adjustment	32
A 3.3.2	Headrest tilt adjustment	32
A 3.4	Right-hand armrest (option)	32
A 4	Operating the dental chair	34
A 4.2	Chair operating elements	36
A 4.2.1	Keypads	36
A 4.2.2	Chair foot control	36
A 4.3	Manual operation of the dental chair	38
A 4.3.1	Dental chair 'up'	38
A 4.3.2	Dental chair 'down'	38
A 4.3.3	Backrest 'up'	38
A 4.3.4	Backrest 'down'	38
A 4.4	Calling automatic chair positions	40
A 4.4.1	LP program	42
A 4.5	Calling automatic positions via the chair foot control	42
A 4.6	Storing automatic chair positions	44
A 4.7	Special chair functions	44
A 4.8	Important note	44
A 5	Operating the foot control	46
A 5.1	Operating elements on the foot control	46
A 5.1.1	Spray preselection and indication	46
A 5.1.2	Instrument priority	46
A 5.1.3	Chip-blower	48
A 5.1.4	Foot control "on"	48
A 5.1.5	Speed control	48
A 5.1.6	Preselecting CCW motor rotation	48
A 6	Operating the dentist's unit	50
A 6.1	Fiber optics	50
A 6.1.1	Fiber optics preselection	50
A 6.1.2	Adjusting the fiber optics intensity	52
A 6.2	Multiflex instrument handling	56
A 6.2.1	Turbine	56
A 6.2.2	INTRAFlex and INTRAFlex LUX instruments	60
A 6.2.3	SONICflex 2000 N dental scaler and SONICflex LUX 2000 L dental scaler	64
A 6.3	Handling of motors and INTRAmatic instruments	66
A 6.3.1	INTRA K LUX motor 192.....	66
A 6.3.2	INTRAmatic motor 181 M/L	68
A 6.3.3	INTRAmatic instruments	70
A 6.4	Sonosoft LUX dental scaler	72
A 6.5	3-function handpiece (abbreviated: 3-F)	74
A 6.6	Spittoon functions	76
A 6.6.1	Bowl rinse on/off	76
A 6.6.2	Special function: timer-controlled bowl rinse	76
A 6.6.3	Special function: spittoon, saliva ejector	76

A 6.6.4	Tumbler filter	78
A 6.6.5	X-ray viewer	78
A 6.6.6	Hydrocolloid.....	78
A 7	Operating the assistant's unit	80
A 7.1	Operating the suction hoses	80
A 7.2	3-function handpiece	80
A 7.3	Spittoon function	80
A 7.4	Hydrocolloid	80
A 7.5	Poly LUX I polymerization light	82
A 7.5.1	Poly LUX II polymerization light	84
A 8	Maintenance and care	86
A 8.1.1	Maintenance and care of motors	86
A 8.1.2	Multiflex coupling	86
A 8.2	Turbine return air filter	88
A 8.3	Maintenance and care of 3-function handpiece	90
A 8.4	Dental scalers	90
A 8.4.1	Sonosoft LUX dental scaler	90
A 8.4.2	SONICflex 2000 N ZEG dental scaler and SONICflex LUX 2000 L dental scaler	92
A 8.5	Poly LUX I	92
A 8.5.1	Poly LUX II	92
A 8.6	Spittoon care	94
A 8.6.1	Disinfecting the suction hoses	94
A 8.6.2	Water-operated saliva ejector (option)	94
A 8.6.3	Suction hose nozzle care	94
A 8.6.4	Suction connector (filter insert) care	96
A 8.6.5	Spittoon and spittoon bowl strainer	98
A 8.6.6	Automatic separator	98
A 8.6.7	Heavy particle collector	98
A 8.7	Supply unit care	100
A 8.7.1	Refilling water disinfectant	100
A 8.8	Overview of maintenance and care for 1060	103
A 9	Troubleshooting	106
A 10	Optional accessories	112
A 10.1	3-function handpiece	112
A 10.2	Polymerization light Poly LUX	112
A 10.3	Hydrocolloid kit	112
A 10.4	Saliva ejector	112
A 10.5	Automatic separator with rinse unit	114
A 10.6	X-ray viewers	114
A 10.7	Dental scalers	114
A 10.7.1	Sonosoft LUX dental scaler	114
A 10.7.2	SONICflex 2000 N / 2000 L dental scaler	114
A 10.8	KAVOLUX 1410	114
A 10.9	Cervical table 1561	114
A 10.10	Right-hand armrest	114
A 10.11	Water boiler	116
A 10.12	Water disinfection system	116
A 10.13	Amalgam separator Multi System Type 1 (kit 1690 A)	116
A 10.14	Chair foot control	116
A 10.15	Water pressure accumulator 1320	116
A 11	Supplementary instructions	118
A 11.1	Sonosoft LUX ZEG (Scaler)	118

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KaVo China
Service and Sales office
Block 2 Huiwan Overseas-
Chinese Apartments
Area 3 of Huianli
Chaoyang District # 12-03
Unit 2
Beijing 100101
Tel. ++ 86 / 10 64 / 91 65 87
Fax ++ 86 / 10 64 / 91 65 87

KaVo Polska sp.z.o.o.
ul. Pomorska 251 P 405
92-214 Lodz
Polska
Tel. ++ 48 / 42 / 6 75 75 36
Fax ++ 48 / 42 / 6 75 75 36

KaVo Osaka
Yotsubashi Okawa Building
11 Fl. 1-6-23, Shinmachi
Nishi-Ku
Osaka 550-0013
Japan
Tel. ++ 81 / 6 / 65 38 58 50
Fax ++ 81 / 6 / 65 38 58 55
e-mail: info@kavo.co.jp

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A 1 User notes

Meaning of symbols

 *Situation which, if the instructions are ignored, may lead to danger, damage to material or operating faults.*

 *Important information for operators and technicians.*

A 1.1 Important notes

 *KaVo SYSTEMATICA® 1060 dental treatment unit to ISO 74 94 and dental treatment chair to ISO 68 75*

The operating, maintenance and installation instructions should be read by the operator/user before commissioning, in order to avoid incorrect operation and damage. If other language versions are required, please contact your responsible KaVo agent.

Duplication and disclosure of the operating, maintenance and assembly instructions (OMA) are subject to prior consent by KaVo.

All the technical data, information and characteristics of the equipment described in these OMA instructions were compiled to the best of our knowledge and reflect the status at the time of printing.

KaVo shall not be liable for damage resulting from

- external influences (low media quality or improper installation)
- application of incorrect information
- usage for purposes other than the intended purpose
- improper execution of repairs

KaVo products may be repaired and serviced by

- the technicians of KaVo's subsidiaries all over the world
- technicians employed by authorized KaVo dealers and specially trained by KaVo
- independent technicians specially trained by KaVo

Approvals expire upon modification by third parties.

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Responsibility is assumed for the safety, reliability and performance of the unit provided that

- assembly, extensions, readjustments, modifications or repairs are carried out by persons trained by KaVo and the assembly was reported to KaVo by means of a transfer protocol,
- the electrical installation of the relevant room meets the requirements and specifications of DIN VDE 0100-710.
- and the unit is operated in accordance with the operating instructions.

 It is advisable to use only original KaVo spare parts for operation and for repair since these have been subjected to comprehensive testing with regard to their safety, function and specific suitability.

Statutory provisions

The general guidelines and/or national laws, national regulations and the rules of industry relating to medical products must be applied to the KaVo product in accordance with the prescribed purpose and must be fulfilled for commissioning and during operation.

A 1.1.1 Safety checks

- The KaVo 1060 unit system is not approved for operation in areas where there is a risk of explosion.
- Each time before using the unit, the user must be certain of the operational safety and the proper state of the unit.
- It is not permitted to continue working with damaged functional parts.
- Instrument tubes may not be stuck together.
- Instrument tubes must be subjected to regular visual inspection and must be replaced immediately if they are damaged.
- Repairs to instrument tubes may be performed only by the persons mentioned in Section A 1.1.
- The insufflation of spray air or blown air into open wounds in the operation area must be avoided since otherwise there is a danger of air embolism or cutaneous emphysema.
- Owing to stagnation, water-carrying or air-carrying lines in treatment units must be flushed through or blown through on commissioning or after periods of nonoperation (weekend, public holiday, holiday, etc.). Remove every handpiece/motor (without mounted instrument) from the support and operate alternately with water and air.
- Carry out intensive disinfection
- Activate the mouthwash glass filler several times.

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- After the end of the treatment, remove rotating instruments from turbines, straight and contra-angle handpieces and treatment heads.

- Never replace scalers without the key - danger of injury and infection.

- Switch off the main switch before leaving the practice.

- After maintenance work, interventions and repairs on the unit, the unit must be made ready for operation according to VDE 0751-1 (according to the state of the art) before being put into operation again.

Further annual measures:

Visual inspection, based on DIN EN ISO 6875, 7494 and 60601-1, of:

- Stability of the unit with regard to standing and tilting
- Proper movement sequences of the components of the unit
- Stability and mechanical intactness of the spring arms
- Intactness of the housing claddings and covers
- Penetrated moisture and scuff marks on the connections

During the maintenance work, the service engineer should check all safety devices:

- Braking and locking devices
- Settings and final control elements of moving parts
- Safety equipment of the customer's water connection according to DIN 1988-4 and prEN 1717

Interference on electro-medical devices from mobile telephones

To guarantee proper operation of electro-magnetic devices, we recommend prohibiting the use of mobile telephones in dental offices, surgeries and clinical areas.

Patients with cardiac pacemakers

Dental equipment may affect the function of implanted pacemakers. Please note the relevant information provided in the trade journals.

Usually, patients with pacemakers will indicate before each treatment that they are wearing an electrical cardiac stimulator. Looking at the pacemaker identity card will allow the experienced dentist to determine the influence his electrical equipment may have on the pacemaker. It is recommendable, particularly in the case of older patients, to inquire whether they are wearing a pacemaker when asking the obligatory questions regarding previous diseases, tendency to hemorrhages, etc. to obtain their history.

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A 1.2 Equipment cleaning and maintenance

Due to the many kinds of medicaments and chemicals used in the dental surgery, upholstery or paintwork surfaces as well as plastic parts may become damaged.

Research has shown that surfaces cannot be protected 100 % against all the products on the market.

Since damage to surfaces caused by these products depends largely on the time taken to react, it is imperative that the places affected be immediately wiped with a damp cloth.

Possible residues of disinfectants may be partially removed from paintwork and plastic surfaces using neutral, non-abrasive detergents and cleansers.

Use only water and mild, non-abrasive cleansers to clean new paintwork surfaces which still cause formation of beads of water.

Worn paintwork surfaces, recognizable by reduced brightness, darkening or cloudy colors, are first cleaned as described above, then preserved using KaVo paintwork preserver 2090 (order no. 489 3418). Apply the paintwork preserver in a circular motion using a lint-free cloth. Then polish with cotton wadding or cloth to obtain a shiny surface. Use of other cleansers may result in damage to paintwork.

Upholstery and plastic surfaces should be cleaned with KaVo Elastoclean spray II, order number 0.489.3467 (0.5 litre) or order number 0.489.3469 (1 litre).

 Make sure during cleaning that fluids do not penetrate into the equipment since this may cause malfunction.

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A 1.3 Equipment disinfection

Use of alcoholic disinfectants

The use of alcoholic disinfectants is subject to the **essential requirement that the disinfectants are used in accordance with the manufacturer's instructions.**

Disinfectants can be obtained on the market in a great variety of concentrations.

As a guideline in the use of disinfectants, we give below the constituents contained in the substances tested with positive evaluations.

The values given here are maximum values and must **not** be exceeded!

Ethanol 96 %
= max. 40 g per 100 g disinfectant

Propanol
= max. 35 g per 100 g disinfectant

Glutaric dialdehyde 25 %
= max. 75 mg per 100 g disinfectant

Ethyl texanal
= max. 10 mg per 100 g disinfectant

Formaldehyde solution
= max. 10 mg per 100 g disinfectant

Glyoxal
= max. 165 mg per 100 g disinfectant

KaVo will not guarantee the use of substances whose constituents deviate from those listed above.

A 1.4 Disinfection of instruments

The KaVo products

- STERImaster®
- LIFEtime®
- Sprayrotor

are best suited to maintain, sterilise and disinfect KaVo instruments.

 *Proceed according to the instructions supplied with the instrument in question.*

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A 1.5 Improper use

Motors and turbines as well as handpieces and contra angle handpieces have been designed exclusively for dental treatment.

Technical work requiring greater pressures than buccal treatment, for example, grinding work on dentures, etc., necessitates the use of special machines for technical use equipped with heavy duty bearings.

Bur change

On turbines, contra angle handpieces and instruments with press button chuck systems, the button may only be pressed when the bur is at a complete standstill.

Otherwise the chuck will be damaged and no longer fit for use.

Using damaged chucks may result in injury due to burs coming loose.

The burs should therefore be checked for proper seating before treatment and after every bur change.

Chuck defects due to incorrect bur changes are easily recognizable and shall preclude any claims of recourse or warranty claims.

Dental chair

The maximum load of 135 kg for the upward movement must not be exceeded.

Tray holder load

Maximum tray holder loads:

KaVo SYSTEMATICA® 1060 S	2 kg
KaVo SYSTEMATICA® 1060 T	3 kg
KaVo SYSTEMATICA® 1060 G	3 kg

These maximum loads must not be exceeded.

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A 1.6 Customer service

Technical service for KaVo equipment is provided primarily by the KaVo agent.

The technicians employed by the KaVo agent and the KaVo customer service technicians continuously participate in general and special training courses at the factory, and are familiar with the entire KaVo product line.

KaVo equipment should be serviced at the recommended intervals in order to guarantee continued operability and maintain the value of the equipment.

Maintenance and servicing include all the checks prescribed by the MPG (medical products law/VDE 0751)* that are to be performed at specific intervals.

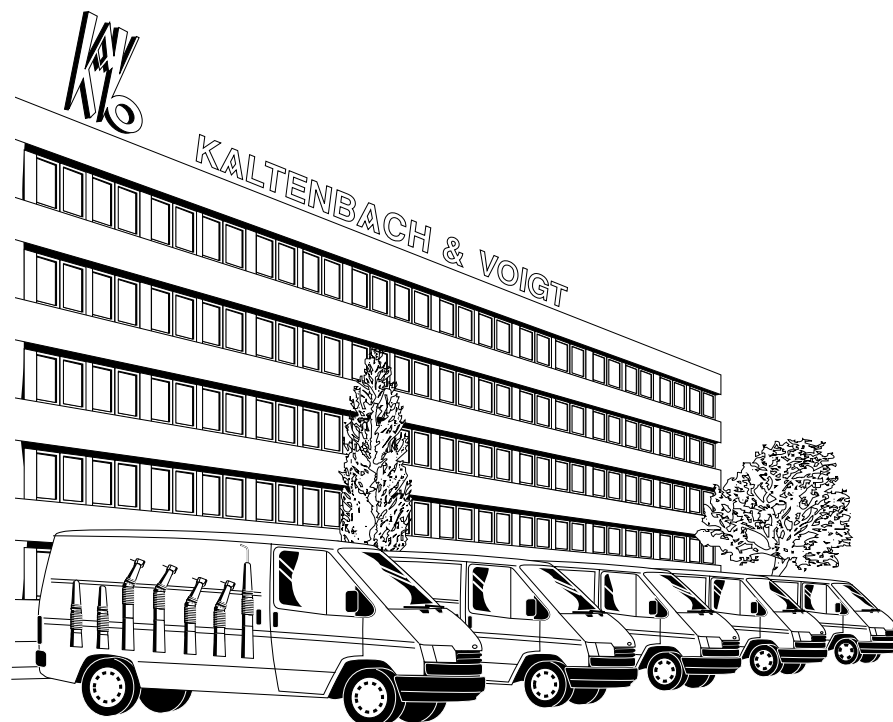
*Applies to Germany only.

Please contact us at the following addresses regarding your inquiries:

KaVo Dental LTD.
Industrial Estate
Raans Road
Amersham Bucks. HP6 6JL
TF 0044/494/721521

KaVo Warthausen
Bahnhofstraße 20
✉ D-88445 Warthausen
📦 D-88447 Warthausen
Federal Republik of Germany
TF 07351/56-0

Always specify the unit type and serial number with your inquiries.

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Tablette de service 1561 = plateau instruments
(228 3024)

A 2.2 Dentist's unit

- ① Table height adjustment brake
- ② X-ray viewer
- ③ Control keys and indicators
- ④ Tray holder
- ⑤ Turbine
- ⑥ INTRA K LUX motor or air motor (depending on order)
- ⑦ Dental scaler
- ⑧ 3-function handpiece

The arrangement of the instruments can be changed as required. This does not apply to the 3-function handpiece. It must always be installed in the following locations:

- 1060 T/TK and 1060 G/GK: extreme right position
- 1060 S/SK: extreme left position

With the exception of the 3-function handpiece, the instruments are controlled by the so-called "priority switching function 2. If, for example, the 1st motor is operating, the 2nd motor can be removed from its holder without bur rotation. This priority switching function always applies for the sequence from left to right.

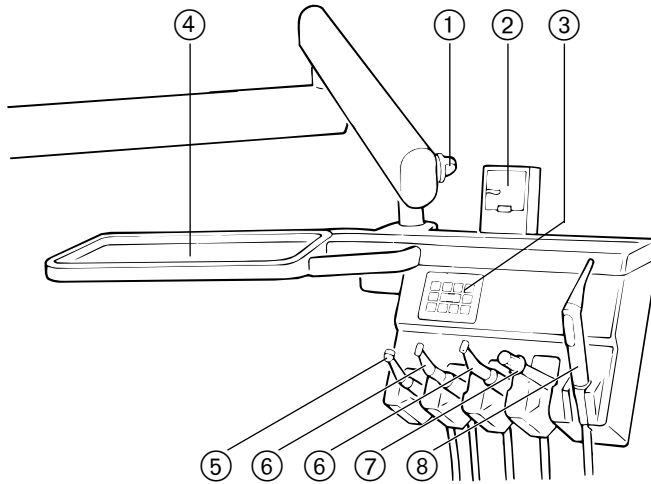
For protection of the tray support, KaVo recommends the following:

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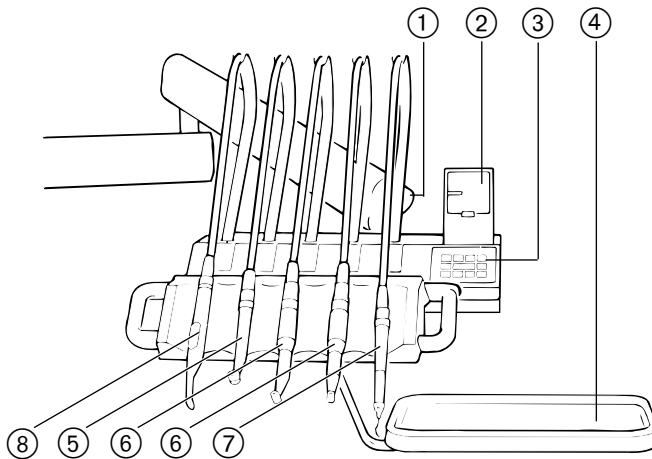
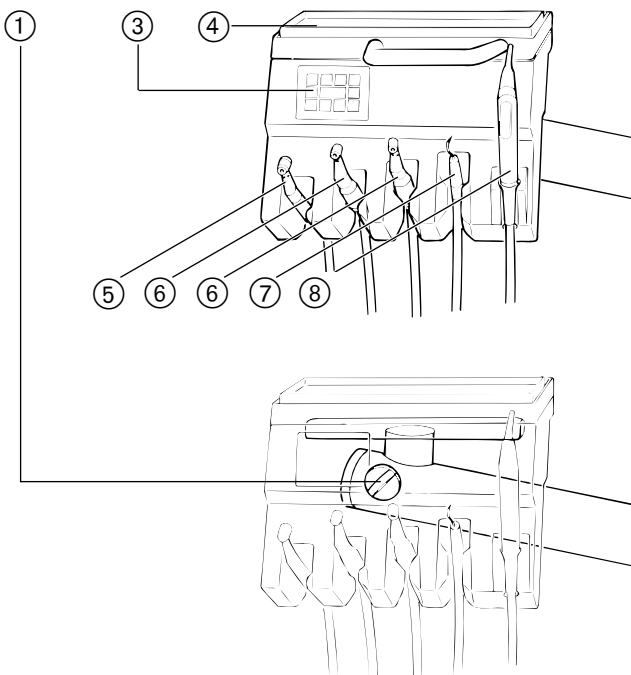
A 2.3 Assistant's unit

- ① Polymerization light
- ② 3-function handpiece
- ③ Spray mist ejector
- ④ Saliva ejector
- ⑤ Hydrocolloid
- ⑥ Control keys and indicators

Always be sure to replace the polymerization handpiece securely in its holder since the light conductor may be damaged if it falls.

A 2.4 Dentist's / assistant's keypad

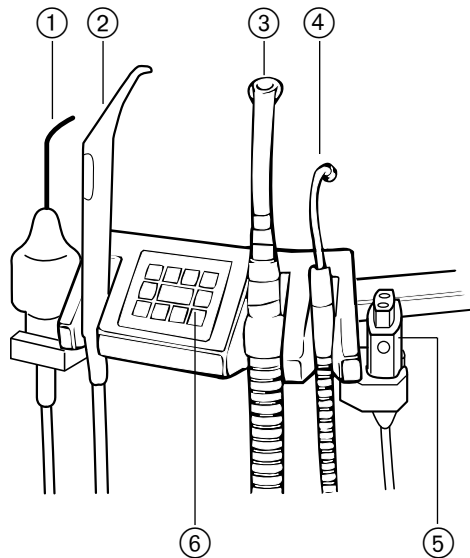
- ⑦ Dental chair up / down
Automatic positions 1 and 2
- ⑧ Backrest up / down
Automatic positions 3 and 4
- ⑨ Fiber optics light on / off
- ⑩ Tumbler filler
- ⑪ Indicator for counterclockwise motor operation
- ⑫ Bowl flush
If hydrocolloid is present: also used for hydrocolloid on / off
- ⑬ X-ray viewer on / off
- ⑭ Spray indicator
- ⑮ Memory key
- ⑯ Preselection key for automatic positions and last position
- ⑰ 'Power on' indicator

Applies to INTRA K motor only.

The keypads on the dentist's unit and the assistant's unit are identical except for the following detail: The assistant's keypad does not have an indicator for counterclockwise motor operation ⑪.

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A 2.7 Location of nameplates

The nameplate provides information on:

- the unit type (Type)
- the serial number (SN)
- the order number (REF)

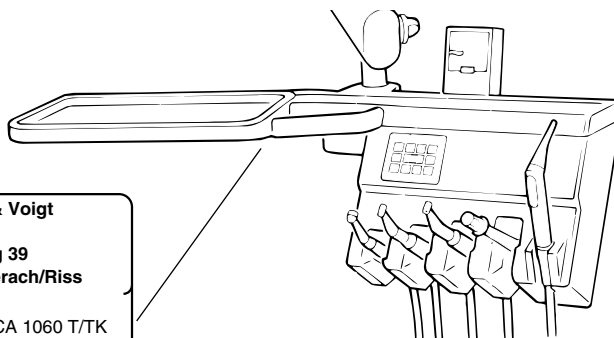
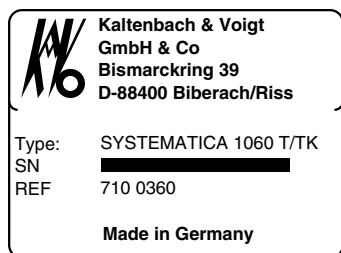
Always specify the unit type and serial number when calling about a problem or a complaint with the KaVo SYSTEMATICA® 1060.

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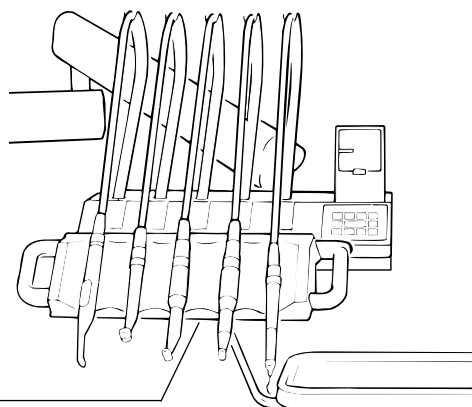
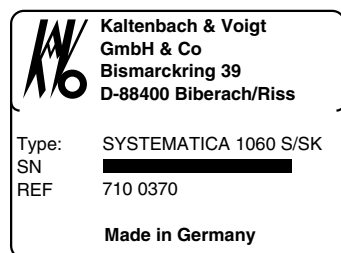
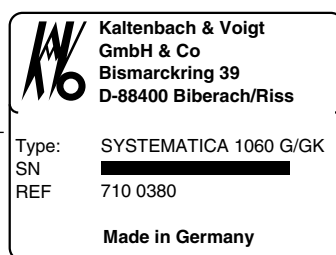
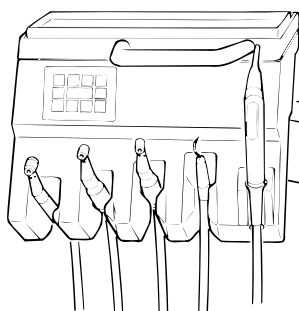
Turn the main switch off before leaving the office.

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The 'on' status is indicated by green LED ⑤ on the dentist's and assistant's units.

A 3.2 Impostazione del programma dimostrativo**A 3.2 Chargement du programme 'Demo' (démonstration)****A 3.2 Loading the demo program**

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- Press and hold foot control pedal ⑥ and simultaneously press and hold program key ④.
- The green LED ③ flashes for approx. 4 seconds.
- Press key ② (AP/LP) during this period.
- Successful storage is indicated by an audible signal.

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This replaces any automatic chair programs and fiber optics intensity previously stored with the demo values.

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A 3.3 Headrest adjustment

A 3.3.1 Headrest height adjustment

The headrest height is adjusted by pulling the headrest out or by pushing it in as indicated by arrows ③.

A 3.3.2 Headrest tilt adjustment

The headrest tilt is adjusted by pressing the locking key ② and tilting the headrest in the directions indicated by arrows ①.

A 3.4 Right-hand armrest (option)

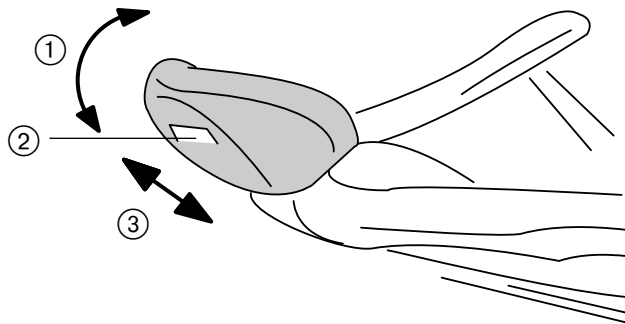
The right-hand armrest can be moved towards the outside, then backward after pressing the release button ④.

When the patient is seated in the chair, move the armrest back in place and lock it in.

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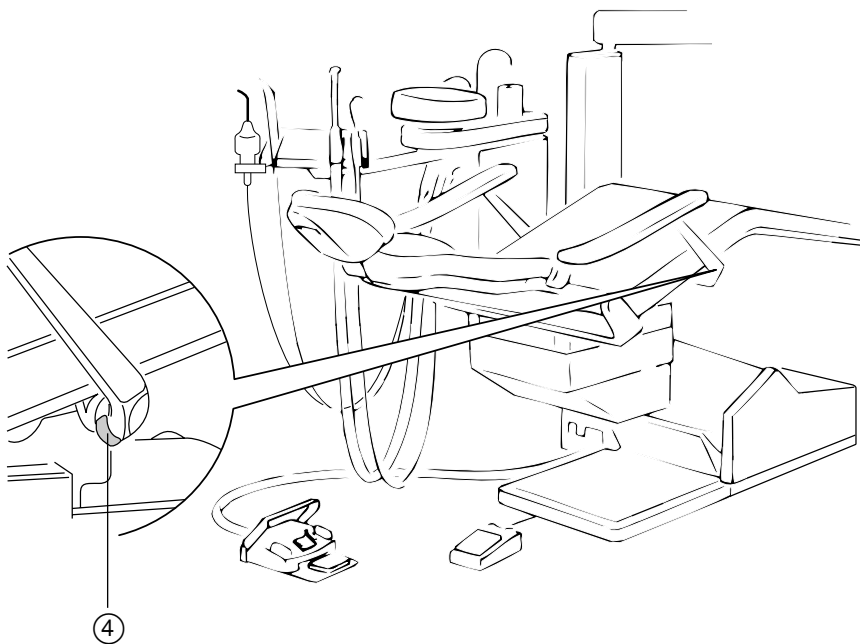
The safety circuits are intended to protect patients, dentists and assistants against injuries.

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If one of safety cutouts ① to ⑤ is actuated, the LEDs in keys 4 - 1 will flash from right to left.

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The safety cutouts immediately terminate the current movement. The chair cannot be repositioned if one of the safety cutouts has been actuated.

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Emergency stop plate ⑥ provides an emergency stop function. If emergency stop plate ⑥ is actuated, all indicators will go off, and any chair movement will stop. Values which have not been stored will be lost.

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Any automatic chair movement can be interrupted by pressing a blue chair control key once more.

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A 4.3 Manual operation of the dental chair

The chair operating keys are blue; the direction of movement is indicated by an arrow on each key.

A 4.3.1 Dental chair 'up'

Press key ① on the dentist's or assistant's unit

or

push the chair foot control plate forward in direction ⑤ (provided your system is equipped with a chair foot control).

⇨ The chair moves up.

A 4.3.2 Dental chair 'down'

Press key ② on the dentist's or assistant's unit

or

push the foot control plate backwards in direction ⑦ (provided your system is equipped with a chair foot control).

⇨ The chair moves down.

A 4.3.3 Backrest 'up'

Press key ③ on the dentist's or assistant's unit

or

push the foot control plate in direction ⑥ (provided your system is equipped with a chair foot control).

⇨ The backrest moves up.

A 4.3.4 Backrest 'down'

Press key ④ on the dentist's or assistant's unit

or

push the foot control plate in direction ⑧ (provided your system is equipped with a chair foot control).

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A 4.4 Calling automatic chair positions

The execution of automatic chair programs must be monitored by office personnel.

Four automatic programs can be called via the keypad on the dentist's or assistant's unit.

To call a program, first press the AP/LP key ⑤, then chair control key ①, ②, ③ or ④.

Example: Calling automatic program 1

Briefly touch the AP/LP key ⑤.

The acknowledgment LEDs in keys ① to ⑤ will blink in turn for approx. 4 seconds. Press the AP 1 key (or desired AP key) during this time.

⇨ The chair moves to automatic position 1.

The LED in key ① illuminates and will continue to be lit even after the stored position has been reached.

Suggested automatic program key assignments (these assignments correspond to the demo program; see section A 3.2):

Note

If the LP program is to be used, the rinse position must be assigned to AP1.

AP1 - Rinse position

AP2 - Treatment position, e.g. for treatment in maxilla

AP3 - Treatment position, e.g. for treatment in mandible

AP4 - Get on/get off position

Of course, the keys AP1 to AP4 may be assigned any position desired by the user with the exception noted above.

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A 4.4.1 LP program

LP means 'last position'.

If the patient is moved to the rinse position in the course of treatment by pressing the AP1 key, he or she may be returned to the previous treatment position - which may be an automatic or manual position - after rinsing by pressing LP.

Calling the LP program

Briefly touch key ⑤.

The acknowledgment LEDs in keys ① - ⑤ will flash in turn for approx. 4 seconds.

Press key ⑤ once more during this period. The acknowledgment LED of key ⑤ AP/LP will be continuously lit, even after the LP position has been reached.

The chair moves to the LP position, i.e. the last position it has been in prior to pressing the AP 1 key.

If the LP key is pressed after switching the unit on in the morning, after lunch break or after an emergency stop, the chair cannot move since no previous position has been stored.

A 4.5 Calling automatic positions via the chair foot control

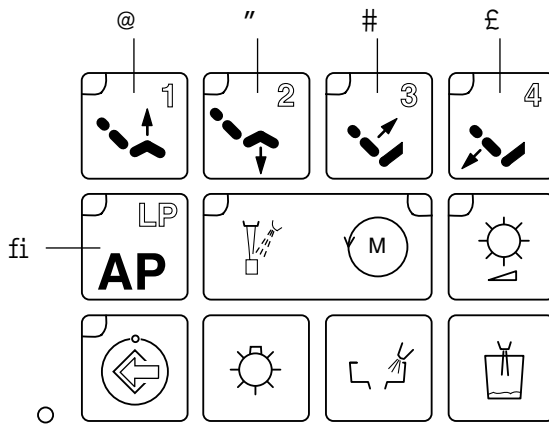
Two automatic programs can be called with the chair foot control.

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A 4.6 Storing automatic chair positions

- Move the chair to the desired position using keys ① to ④.
- Press and hold the white memory key.
- The acknowledgment LED ⑥ in the program key will blink for approx. 4 seconds.
- During this time, you must press the key (①, ②, ③ or ④) to which this position is to be assigned.
- Storage of this position is acknowledged by an audible signal ⑦.
- The LED in the actuated key (①, ②, ③ or ④) will illuminate.

A 4.7 Special chair functions

If the unit is operated at mains voltages above 200 V, parallel spindle motor operation is possible, i.e. the chair and backrest movements in an automatic program are performed simultaneously.

If the unit is operated at mains voltages below 200 V, automatic program chair **movements are executed consecutively by necessity** due to the amount of current consumed.

The corresponding adjustments are made by an authorized technician on the main printed circuit board.

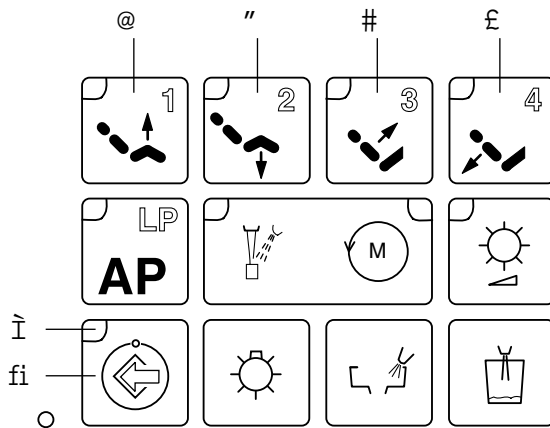
A 4.8 Important note

The chair and backrest motors are protected by thermal releases that switch the motors off at a temperature of 140° C and turn them back on when they have cooled down.

Such temperatures are never reached during normal operation in the surgery.

However, frequent use, particularly of the up and down movements, during demonstrations and exhibitions may increase the operating temperature to the limit for shutoff (after approx. 8 complete motion sequences).

The duration of the cooling phase depends on the ambient temperature; it is typically approx. 15 minutes.

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A 5 Operating the foot control**A 5.1 Operating elements on the foot control**

- ① Spray preselection key
- ② Preselection of CCW motor rotation
- ③ Chip blower
- ④ Instrument 'on'
- ⑤ Speed control

A 5.1.1 Spray preselection and indication**Spray preselection**

- Actuate spray preselection key ①.
- The green LED ⑥ on the dentist's unit or assistant's unit lights up.

Spray action has been preselected for all drilling instruments and for a Sonosoft dental scaler or SONICflex dental scaler if present.

Spray deselection

- Actuate spray preselection key ① once more.
- The green LED ⑥ on the dentist's unit or assistant's unit goes off.

Spray action has been deselected for all drilling instruments and for a Sonosoft dental scaler or SONICflex dental scaler if present.

A 5.1.2 Instrument priority

Among the drilling instruments, there is a priority from left to right.

Example:

You are working with the instrument on the very left. All instruments to the right of this instrument are disabled, even if they are removed from their holders. Thus, it is possible, for example, to change burs on these instruments.

Contrary to this, if you are working with the instrument on the very right, and remove any of the instruments to the left of it, the instrument removed first is disabled, and the instrument removed last is activated.

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A 6 Operating the dentist's unit**A 6.1 Fiber optics****A 6.1.1 Fiber optics preselection**

The fiber optics light can be programmed to be preselected whenever the unit is switched on.

Fiber optics programming includes all drilling instruments and the Sonosoft LUX and SONICflex LUX dental scalers.

Preselect the desired status by pressing the fiber optics key ②.

Fiber optics 'on' - LED ① is lit.

Fiber optics 'off' - LED ① is off.

Storage

Press and hold program key ③.

LED ④ will flash for approx. 4 seconds.

Press the fiber optics key ② during this period.

The unit acknowledges the storage of your selection by issuing an audible signal ⑤.

If there is no audible signal, repeat the programming procedure.

Note

The fiber optics preselection can be manually changed even after it has been stored.

The stored setting will be effective again the next time the unit is powered up.

Manual fiber optics preselection

Of course, it is possible anytime to manually switch the fiber optics on or off.

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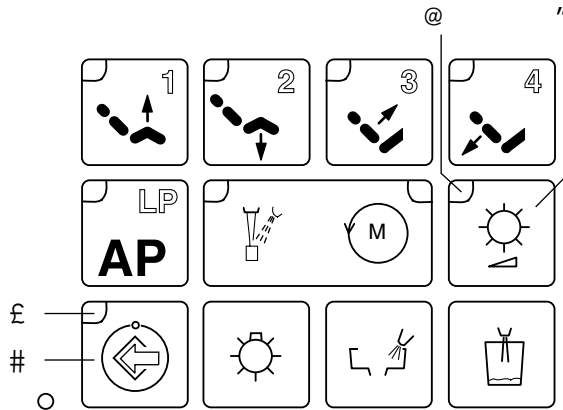
A 6.1.2 Adjusting the fiber optics intensity

There are 8 fiber optics intensity settings.

The desired fiber optics intensity setting may be stored.

This stored fiber optics intensity setting will be used for all drilling instruments and the Sonosoft LUX and SONICflex dental scalers.

Adjustment

- The fiber optics must be preselected, i.e. the green LED ① must be lit.
- Briefly press memory key ③.
- LED ④ will blink for approx. 4 seconds.
- Press the fiber optics key ② during this period.
- LED ④ will be continuously lit.
- The fiber optics key ② is used to cycle the fiber optics intensity through the 8 settings.

The intensity is indicated by the green LED ①.

- ⇨ At maximum intensity, LED ① is continuously lit, at minimum intensity it blinks very slowly.
- ⇨ The settings in between are indicated by different blink rates.
- Press the fiber optics key as often as required to obtain the desired intensity setting.
- ⇨ LED ① will be continuously lit or will blink as described above.

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A 6.2 Multiflex instrument handling

A 6.2.1 Turbine

Plug the Multiflex coupling 465 R / 465 LR into the turbine hose and secure with the union nut.

Engaging the turbine

- Insert the turbine axially and latch it in.

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Removing the turbine from the coupling

- Pull the turbine off axially by turning it slightly.

Bur change

(press button chuck system)

- Firmly push down on the press button after the turbine rotor has come to a standstill.
- Remove/insert the bur.
- Release the press button.

Make sure the bur is firmly seated.

Pushing the press button while the turbine is still rotating may destroy the chuck and cause injury.

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A 6.2.2 INTRAFlex and INTRAFlex LUX instruments

Inserting the INTRA head

- Turn the tension ring in the direction indicated by the arrow and hold it in this position.
- Insert the INTRA head, turn the tension ring in the opposite direction and tighten well.

Make sure the INTRA head is firmly seated.

Engagement

- Insert the INTRAFlex instrument axially and latch it in.

Non-axial insertion during the fiber optics shutoff delay period may destroy the high-pressure lamp.

Removing the INTRAFlex instrument from the coupling

- Pull the INTRAFlex instrument off axially by turning it slightly.

Bur change

(press button chuck system)

- Firmly push down on the press button after the bur has come to a standstill.
- Remove/insert the bur.
- Release the press button.

Make sure the bur is firmly seated.

Pushing the press button while the instrument is still rotating may destroy the chuck and cause injury.

Bur change

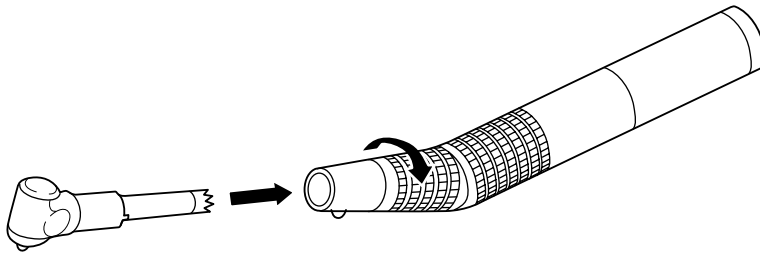
(conventional chuck)

Burs/grinders are inserted and ejected using a bur changer.

Make sure the bur/grinder is firmly seated.

Rubber polishers or brushes are screwed on or pushed onto an interchangeable spindle head.

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A 6.2.3 SONICflex 2000 N dental scaler and SONICflex LUX 2000 L dental scaler

Engagement

- Insert the instrument axially into the Multiflex coupling and latch it in.

Removing the SONICflex 2000 N or SONICflex LUX 2000 L from the coupling

- Pull the instrument off the Multiflex coupling axially by turning it slightly.

Changing instrument tips

A special changing tool is supplied for changing the instrument tips.

To prevent injury, the changing tool must remain attached to the tip when not in use.

An instrument tip is inserted in this special wrench and then screwed onto the instrument by turning it clockwise until hand-tight.

When the dental scaler is operated, the instrument tip will automatically tighten itself to the required degree of tightness.

To remove a tip from the instrument, insert the empty special wrench over the instrument tip and unscrew by turning it counterclockwise.

Inspect instrument tips for proper operating condition.

Operating the SONICflex dental scaler

- Remove the dental scaler from its holder.

1060 S/SK: Pull the swing arm forward until the microswitch in the module is actuated.

- Start the dental scaler by depressing the foot control pedal ①.
- The instrument intensity is controlled by moving the foot control pedal to the left or right as indicated by arrows ②.
- The black adjustment ring ③ on the coupling allows for infinitely variable spray adjustment.

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A 6.3 Handling of motors and INTRAmatic instruments

A 6.3.1 INTRA K LUX motor 192

- Plug the INTRA K LUX motor 192 into the motor hose axially and secure with the union nut.

The motor and hose coupling are coded to connect in one position only. Avoid damaging contacts and fine thread.

Engaging an instrument

- Push the instrument onto the motor coupling axially all the way to the stop.
- Twist the instrument until it locks in audibly.

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- Start the motor by depressing the foot control pedal ①.
- The motor speed is controlled by moving the foot control pedal to the left or right as indicated by arrows ②.
- The black adjustment ring ③ on the motor hose allows for infinitely variable spray adjustment.

See section A 5.1.6 for changing the direction of rotation.

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A 6.3.2 INTRAmatic motor 181 M/L

i *Fiber optics (LUX) instruments cannot be used with the 181 M motor.*

Engaging an instrument

- Push the instrument onto the motor coupling all the way to the stop.
- Twist the instrument until it locks in audibly (**181 L only**).

Removing the instrument

- Pull the instrument off axially by turning it slightly.

Operation

- Remove the motor from its holder.

i *1060 S/SK: Pull the swing arm forward until the microswitch in the module is actuated.*

- Start the motor by depressing the foot control pedal ①.

Speed control/changing the direction of rotation

- Move the foot control pedal ② to the max. position (to the right).
- By turning the adjustment sleeve ③ or ⑤, the motor speed may be infinitely adjusted between standstill and the maximum speed in the clockwise and counter-clockwise directions of rotation.

i *If the adjustment sleeve is set to the maximum position for one direction of rotation, the speed may be controlled by moving the foot control pedal left or right as indicated by arrows ②.*

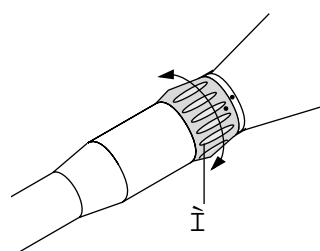
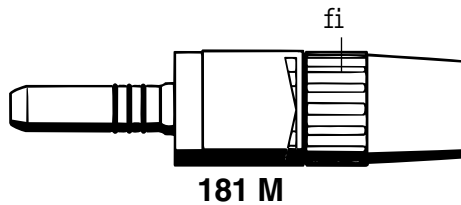
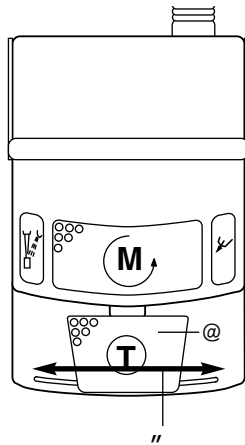
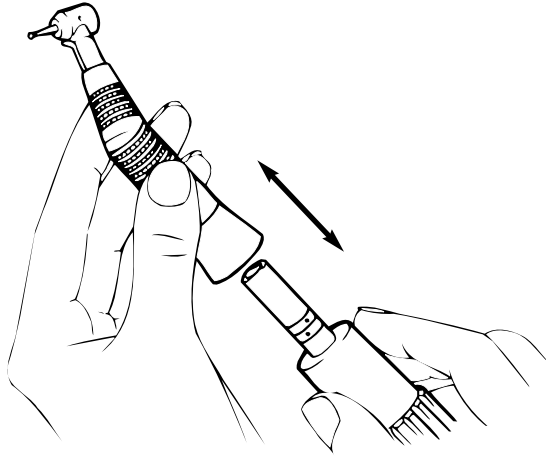
Spray control

i *INTRAmatic motor 181 L*

- Infinitely variable spray control is possible by turning adjustment ring ④ on the motor.

i *INTRAmatic motor 181 M*

- Infinitely variable spray control is possible by turning the black adjustment ring on the Multiflex coupling.

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A 6.3.3 INTRAmatic instruments**Handpiece**

- Turn the tension ring in the direction indicated by the arrow.
- Insert the bur in the chuck.
- Turn the tension ring in the opposite direction.

Make sure the bur is firmly seated.

Contra angle handpiece

- Turn the tension ring in the direction indicated by the arrow and hold.
- Insert the INTRA head. Turn the tension ring in the opposite direction and tighten firmly.

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Bur change

(press button chuck system)

Contra angle handpiece must have come to a standstill before burs can be changed.

- Firmly push down on the press button.
- Insert the bur by turning it slightly until it latches in, or remove the bur.
- Release the press button.

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A 6.4 Sonosoft LUX dental scaler

Insert an instrument tip in the changing tool supplied as part of the accessories and tighten **only slightly** (hand-tight).
Inspect instrument tips for proper operating condition.

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Operation

The fiber optics intensity is adjusted as for the drilling instruments.

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There is **no** blowing air function for the Sonosoft LUX dental scaler on SYSTEMATICA 1060 treatment unit's.

- Preselect spray (see section A 5.1.1).
- Start the instrument by pressing the foot control pedal ③.

The instrument does not start up when the foot control pedal is in the „minimum“ position, i. e. all the way left. Move the foot control to the center position.

- The intensity is controlled by moving the foot control pedal to the left or right as indicated by arrows ④.
- The amount of water ejected can be controlled on the dental scaler handpiece (see ①).

Replacing the high-pressure lamp

- Unscrew the dental scaler handpiece from the instrument hose and pull off.

The lamp may be hot.
- Danger of injury -

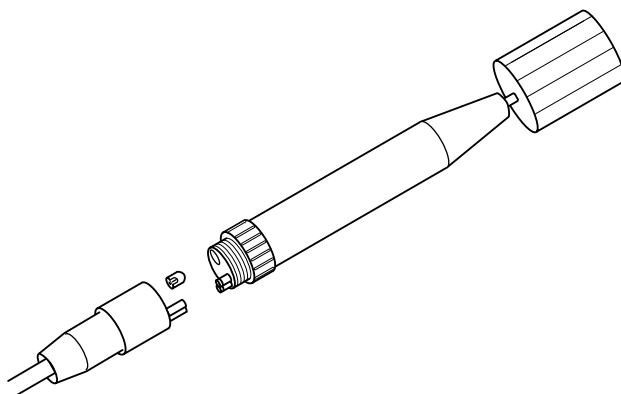
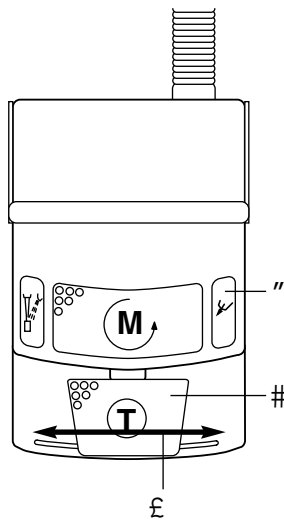
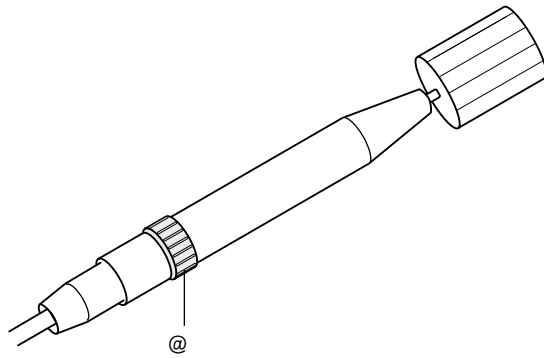
- Pull the high-pressure lamp out of the lampholder and insert a new lamp.

Order no. 553 3881

Make sure contacts are positioned correctly.

- Reinstall the handpiece on the instrument hose.

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A 6.5 3-function handpiece
(abbreviated: 3-F)

Media dosage is infinitely variable.

- For safety reasons, only original KaVo nozzles may be used.
- Snap on the nozzle and check that it is firmly seated.
- Verify media passage through the nozzle before use on patients.

Air

- Press key ③.

⇨ The amount of air expelled can be controlled by increasing or decreasing the pressure on key ③.

Water

- Press key ①.

⇨ The amount of water expelled can be controlled by increasing or decreasing the pressure on key ①.

Spray

- Press rectangular key ②.

⇨ The amount of spray expelled can be controlled by increasing or decreasing the pressure on key ②.

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A 6.6 Spittoon functions

The operating elements (keypads) on the dentist's and assistant's units are identical. This means that the same keys have identical functions.

A 6.6.1 Bowl rinse on/off

When the hydrocolloid option is installed, knurled screw ② must be turned clockwise all the way.

Press key ① to switch on the bowl rinse function; press the key once more to switch off the function.

A 6.6.2 Special function: timer-controlled bowl rinse

Can be set by a technician upon request.

When the hydrocolloid option is installed, knurled screw ② must be turned clockwise all the way. (Also see section A 6.6.6.)

Press key ① to switch on the bowl rinse function; it is automatically disabled after the period set by the technician (max. 51 seconds).

Timer-controlled bowl rinse is mandatory when an amalgam separator is used.

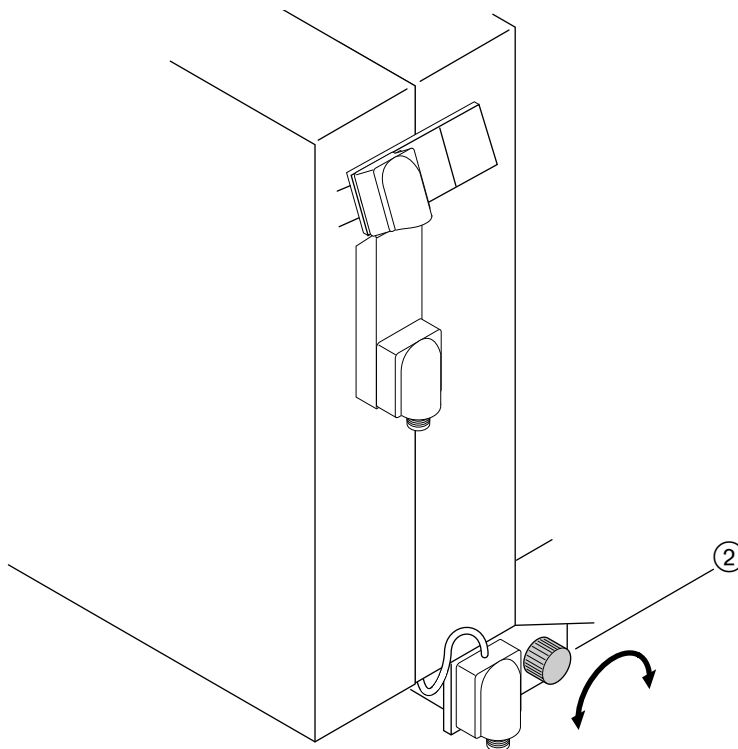
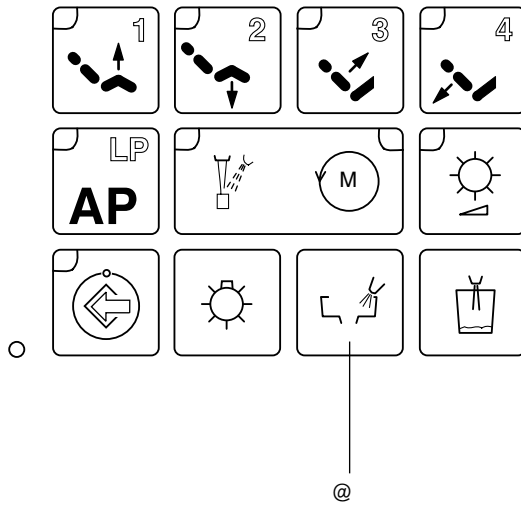
A 6.6.3 Special function: spittoon, saliva ejector

Only recommendable for units with water-operated saliva ejector and low water pressure.

If this function is desired, the unit may be adjusted by a technician such that the bowl rinse function is automatically disabled when the saliva ejector is operated.

This increases the water pressure available for the saliva ejector, thereby increasing its suction.

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A 6.6.4 Tumbler filler

Press key ③ to switch on the tumbler filler. It will be switched off automatically after the period selected at the time the unit is installed.

This period may be adjusted by a technician to a value between 1 and 51 seconds.

A 6.6.5 X-ray viewer (option)

Press key ① to switch on the X-ray viewer. Press the key again to switch the X-ray viewer off.

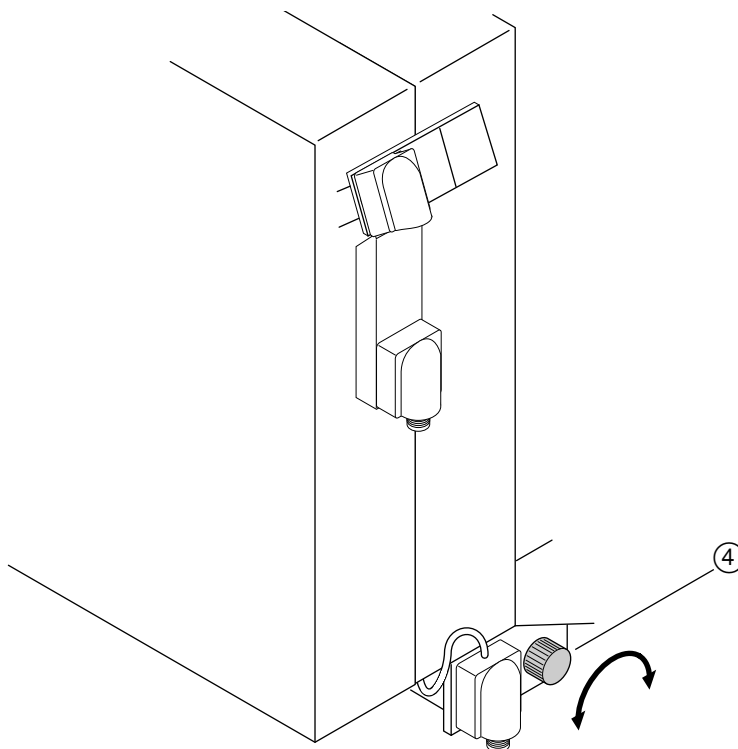
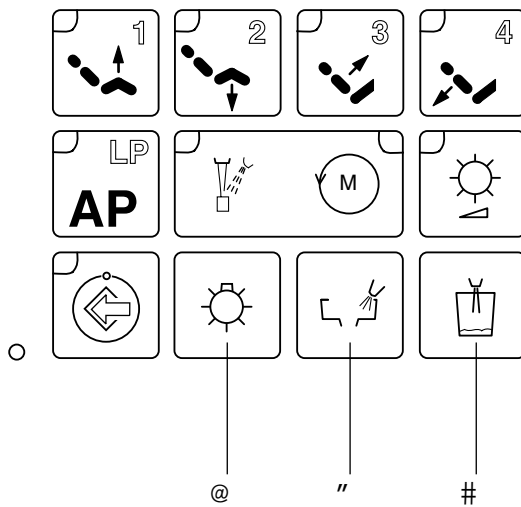
A 6.6.6 Hydrocolloid (option)

- Turn knurled screw ④ all the way counterclockwise.

The hydrocolloid is switched on by actuating bowl rinse key ②. Switch off by pressing the key once more.

If the period for timer-controlled bowl rinse proves to be insufficient, a technician must reinstate the manual bowl rinse on/off function.

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These functions are switched on automatically when the ejectors are removed from their holders.

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The amount of vacuum for spray mist and saliva ejection may be reduced or completely shut off by means of the slides integrated in the conical sections.

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The water-operated saliva ejector does not have the integrated slide in the conical section.

A 7.2 Manipolo a 3 funzioni**A 7.2 Seringue 3 fonctions****A 7.2 3-function handpiece**

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A 7.5 Poly LUX I polymerization light

The illumination period is set to approx. 10 seconds at the factory.

The illumination period can be adjusted by a KaVo-authorized technician in the range from a manual on/off function of the switching taper ① to a shutoff delay of approx. 60 seconds.

Operation

Always observe composite manufacturer's specifications.

- Remove the polymerization handpiece from its holder.
- The polymerization handpiece can be switched on or off using the entire circumference of switching taper ①.

Do not look directly into the light. Longterm danger of injury.

The polymerization light can be switched off before the illumination period is over using switching taper ① on the polymerization handpiece.

When the light is switched on the next time with switching taper ①, the illumination period is again 10 seconds.

The light emitting surface should be equipped with a spacer ring ③ to protect the handpiece against contamination by filling material not yet polymerized.

The polymerization handpiece should be held such that the light emitting surface is approx. 1 to 2 mm above the surface of the filling.

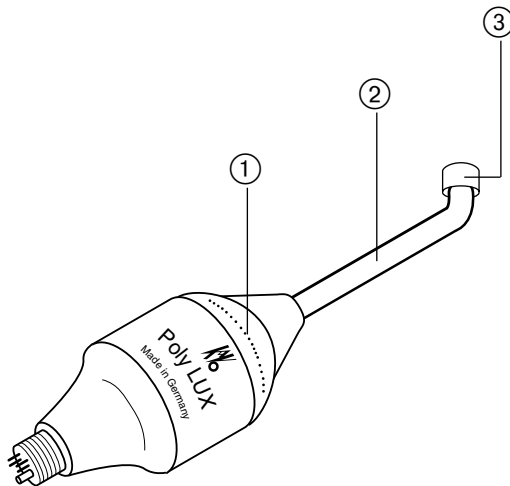
To obtain optimal polymerization results, the light should be started only when the light emitting surface is above the filling to be polymerized.

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See the separate Operating, Maintenance and Assembly instructions (OMA) supplied with the instruments regarding instrument maintenance and care.

A 8.1.1 Manutenzione dei motori**A 8.1.1 Entretien du moteur****A 8.1.1 Maintenance and care of motors**

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Weekly visual inspection of O-rings on the motor coupling for:

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See the separate Operating, Maintenance and Assembly instructions (OMA) supplied with the motors regarding further notes for maintenance, care and sterilization.

A 8.1.2 Giunto Multiflex**A 8.1.2 Accouplement Multiflex****A 8.1.2 Multiflex coupling**

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Weekly visual inspection of O-rings on the Multiflex coupling for:

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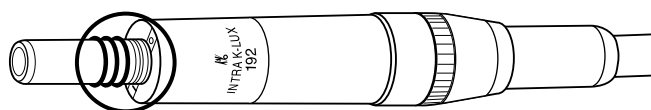
See the separate Operating, Maintenance and Assembly instructions (OMA) supplied with the Multiflex coupling regarding further notes for maintenance, care and sterilization.

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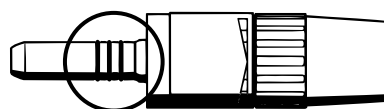
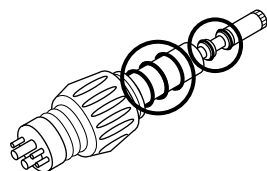
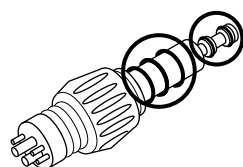
RÉF. 1060 T/G= 220 7926
 1060 S = 200 3098

A 8.2 Turbine return air filter

- Check oil trough every week.
- Replace return air filter as necessary and clean filter holder.

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A 8.3 Maintenance and care of 3-function handpiece

Daily

- Use the supplied nozzle needle ① to clean the water nozzle when clogged.
- Pull nozzle ② off the 3-function handpiece by turning it slightly.

Nozzles and handpieces may be sterilized in an autoclave at up to 135 °C.

Do not sterilize in a hot-air sterilizer or thermodisinfector. Do not place the nozzle and handpiece in a disinfection or ultrasonic bath.

Every evening

- Blow out the water duct in the nozzle.

As required

The O-rings must be replaced when water is ejected along with air.

- Pull off nozzle ② by turning it slightly.
- Replace the O-rings (3):
 - 1 small O-ring (order no. 200 6060)
 - 2 large O-rings (order no. 200 6164)

A 8.4 Dental scalers

A 8.4.1 Sonosoft LUX dental scaler

Scaler tips can be sterilized at up to max.:

- 180 °C (sealed in bags)
- 135 °C (with torque wrench ③ or sealed in bags)

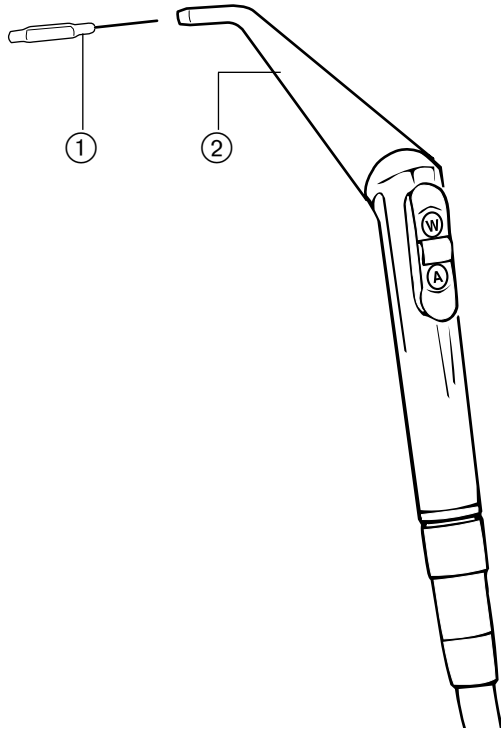
The tips must not be sterilized loose together with other metal parts because contact corrosion may then result.

The dental scaler handpiece **cannot** be sterilized.

The torque wrench ③ serves the following purposes:

- To install and unscrew the tips from the instrument.
- To store the tips.
- To sterilize the tips.
- To protect the tips stored in the changing tool.
- To protect the user and other persons against injury.

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A 8.4.2 SONICflex 2000 N dental scaler and SONICflex LUX 2000 L dental scaler

See the separate Operating, Maintenance and Assembly instructions (OMA) supplied with the SONICflex dental scalers regarding maintenance and care.

A 8.5 Poly LUX I

The assembled Poly LUX handpiece can be disinfected.

Proceed as follows for individual parts:

- Pull off light conductor ④ and unscrew gripping sleeve ③.
- The spacer ring ⑤, light conductor ④ and gripping sleeve ③ can be sterilized in an autoclave at up to 135° C.
- Clean the optics ② with a suitable cloth.
- Connector ① may be wiped with a damp cloth (not using disinfectant) and dried with a lint-free cloth.

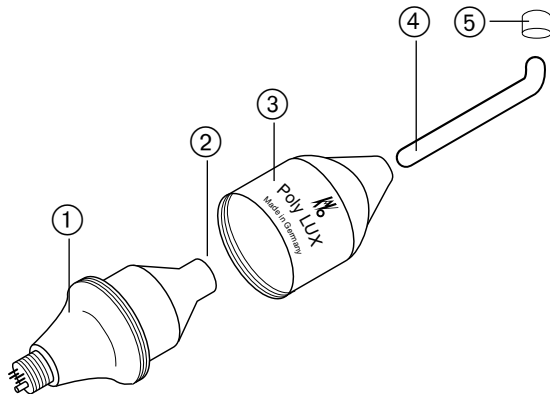
Check at regular intervals whether the various materials are properly polymerized by polymerizing material samples.

Observe the Heliotest instructions (included in accessories) to verify the curing performance.

A 8.5.1 Poly LUX II

For product care, please see the separate instructions for Poly LUX II, Order No. 789 0663.

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A 8.6.1 Disinfecting the suction hoses

Daily after office hours

- Disinfect the suction hoses with a suitable disinfectant. For example, a disinfectant useable for all KaVo units is DEKASEPTOL Gel.

See the separate Operating, Maintenance and Assembly instructions (OMA) supplied with DEKASEPTOL Gel for further notes regarding KaVo DEKASEPTOL Gel.

A 8.6.2 Water-operated saliva ejector (option)

Actuate backflush **several times a day**.

- Pull off nozzle.
- Hold saliva ejector hose in spittoon bowl.
- Pull backflush lever.

A 8.6.3 Suction hose nozzle care

Disinfect the suction hose nozzles according to the instructions supplied with the nozzles.

Daily or as required

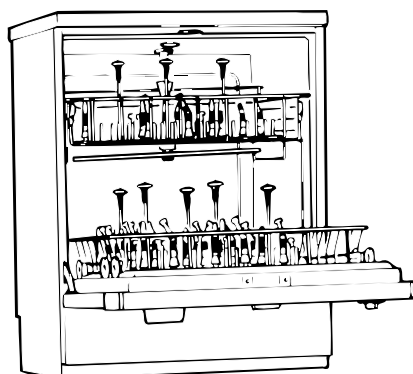
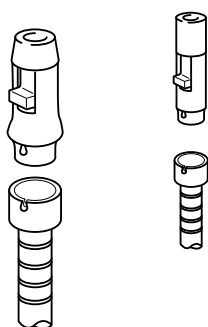
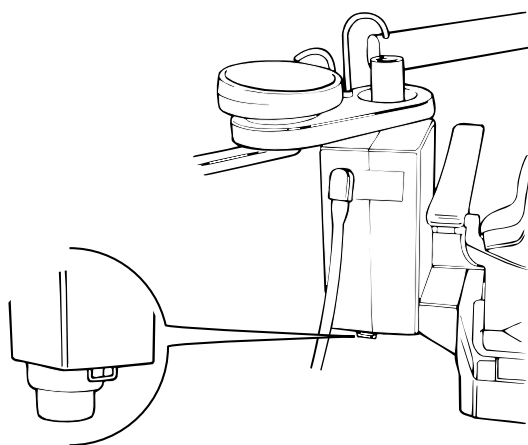
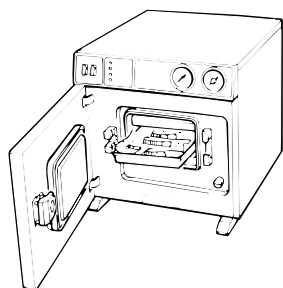
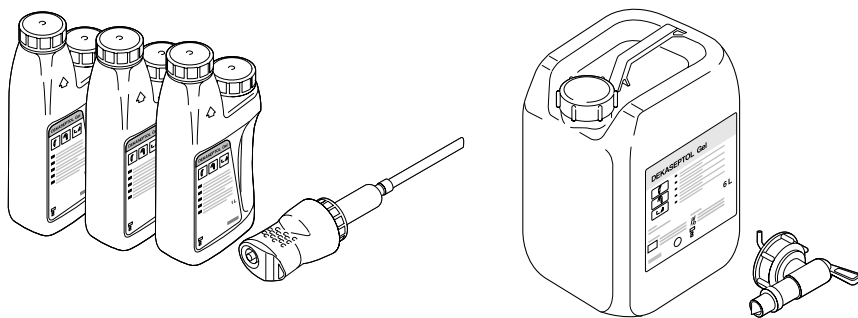
- Pull the nozzles off the suction hoses.

The rubber mouth piece can be unscrewed when a water-operated saliva ejector is present.

- Clean the nozzles by brushing them under running water.

The suction hose nozzles may be washed in a thermodisinfector at up to 95° C.

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A 8.6.4 Suction connector (filter insert) care

- Check filter inserts after surgical treatment, otherwise daily.

Replace filter inserts as required, at the latest when suction decreases.

Cleaning cannot be recommended for reasons of hygiene.

Checking the filter inserts with the vacuum-operated saliva ejector

- Switch the unit on.
- Remove the suction hoses from their holders.
 - ⇨ The suction motor will start up.
- Pull the suction hose elbows out of the suction connector axially and place in the spittoon bowl.
- Pull the strainers ① out of the holders.
Order no. 763 3423

Use a paper napkin under the inserts to prevent dripping.

- See note **'As required'** below.

Checking the strainer with the water-operated saliva ejector

- Switch the unit on.
- Remove the saliva ejector hose from its holder.
 - ⇨ The suction motor will start up.
- Push down holder ④ until it latches in (approx. 1 cm).
- Pull saliva ejector hose ③ off towards the bottom and place in the spittoon bowl.
- Pull strainer ② off towards the bottom.
Order no. 210 2922

Use a paper napkin under the strainer to prevent dripping.

As required

- Empty or replace strainers and dispose of properly.
- Wipe suction hoses with disinfectant.

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A 8.6.5 Spittoon and spittoon bowl strainer**Several times a day**

- Clean the strainer.
- Clean the spittoon bowl according to the instructions supplied with the DEKASEPTOL Gel package.

Do not use cleansers which cause foaming or cleansers containing acid or sand.

A 8.6.6 Automatic separator

The automatic separator itself is **maintenance-free**.

In order to prevent blood and secretions from settling, aspirate a glass of cold water after each treatment.

A 8.6.7 Heavy particle collector**Weekly or as required**

- Check heavy particle level in heavy particle collector.

Procedure

- Engage the tool (from the accessories) at the bottom of the collector bowl and loosen by turning it counterclockwise.
- Lower the collector bowl vertically.
 - ⇨ Clean bowl if required (dispose of contents properly) and reinstall.

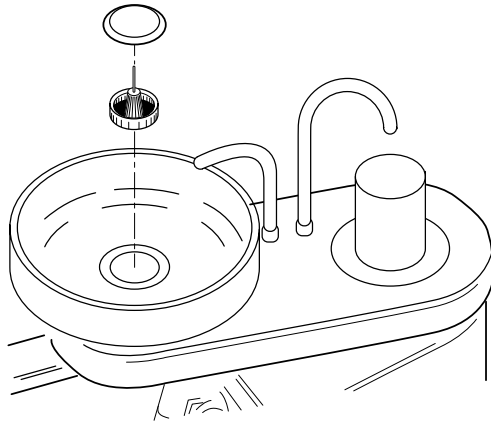
It is absolutely necessary to drain the water remaining in the collector bowl before reinstalling it to prevent overflow.

Only hand-tighten the collector bowl (do not use the tool).

Since tightness is provided by a radially sealing O-ring, reinstalling the collector bowl requires little force.

Overtightening may cause problems when trying to remove the collector bowl the next time.

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A 8.7 Supply unit care**A 8.7.1 Refilling water disinfectant**

Only Oxygenal 6 (**order no. 489 3451**) may be used in KaVo water disinfection systems.

Please observe the expiration date printed on the bottle.

Oxygenal 6 **must** be stored in a dark, cool place.

Consumption of Oxygenal depends on water consumption during treatment.

If the bottle is empty, this is indicated by an audible signal issued at intervals of approx. 10 seconds.

Refilling procedure

- Switch off the unit.
- Remove the lid from the supply unit cover.
- Lift the bottle out vertically.
- Unscrew the lid by turning it counter-clockwise.
- Fill with Oxygenal up to the mark.
- Reassemble in reverse sequence.
- Switch on the unit.

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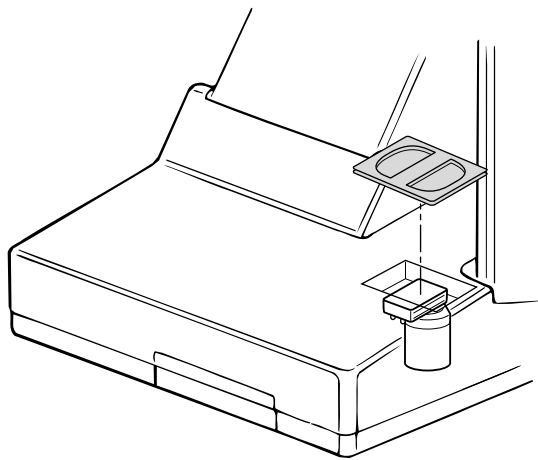
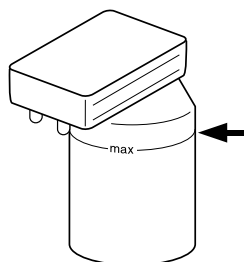
A 8.8

What	When	How	Description in OMA instr.
Strainer in spittoon bowl	several times a day	Clean strainer under running water. Dispose of contents properly.	Section A 8.6.5
Suction hoses	After each treatment	Aspirate a tumbler full of cold water Do not aspirate foaming cleansers.	Section A 8.6.1
Water-operated saliva ejector	In the evening	Disinfect suction hoses with KaVo DEKASEPTOL Gel. Observe instructions included in DEKASEPTOL Gel package.	Section A 8.6.2
	Several times a day	Actuate backflush.	
Strainers in suction connector	After surgical work, otherwise daily	Check strainers. Replace as required, at the latest when suction decreases. Cleaning cannot be recommended for reasons of hygiene.	Section A 8.6.4
Instruments	After each treatment	Change, clean, lubricate and sterilize or disinfect outer surfaces. 2122 ROTA Spray - order no. 411 9850 2142 ROTA Spray 2 - order no. 411 7510 KaVo Spray - order no. 411 9830	Note OMA instructions of instruments
	At noon and in the evening	Lubricate. Allow instruments to dry well on instrument stand..	
INTRA K-Lux motor 192	After each treatment	Disinfect exterior.	Note OMA instructions of motors
INTRAmatic motor 181 L	After each treatment	Disinfect exterior.	
	At noon and in the evening	Lubricate.	
INTRAmatic motor 181 M	After each treatment	Disinfect exterior.	
	At noon and in the evening	Lubricate.	
	As required	Sterilize	
Heavy particle collector	Weekly or as required	Open heavy particle collector with tool (order no 411 1843, from accessories) and clean. Dispose properly of amalgam. Drain water before reinstallation even if bowl does not need to be cleaned to prevent overflow when opening the next time. Only hand-tighten bowl (do not use tool).	Section A 8.6.7
Paintwork surfaces	Monthly or as required	Wipe surfaces with KaVo paintwork preserver (order no. 411 1843) and a soft cloth. Do not use abrasive cleansers or cleansers containing acid. See sections A 1.2-1.3 regarding care and disinfection.	
Upholstery	Monthly or as required	Use KaVo Elastoclean II (order no. 489 3467 = 0,5 l; 489 3469 = 1,0 l) and wipe upholstery off with a soft cloth. See sections A 1.2-1.3 regarding care and disinfection.	
Spittoon bowl	Daily or as required	Clean spittoon bowl according to instructions supplied with Aseptoclean package.	
KAVOLUX 1410	As required	Wipe the reflector with water and a soft cloth when the reflector is cold . Do not use abrasive cleansers or cleansers containing acid. Do not spray disinfect.	Note OMA instructions for KAVOLUX 1410
Amalgam separator Multi System Type 1	After each treatment	Aspirate 1 tumbler (approx. 100 ml) of cold water into suction hoses used. Rinse spittoon bowl.	Note OMA instructions for Multi System Type 1 amalgam separator
	Daily	Check filter drawer and clean as necessary.	
	In the evening	Use DEKASEPTOL Gel according to instructions supplied with DEKASEPTOL Gel package.	

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A 9 Troubleshooting

This section of the operating instructions contains guidelines for troubleshooting by the operator.

If problems cannot be localized and eliminated by following these instructions, a technician authorized by KaVo must be called for repair.

S • Nothing works**C • Main switch switched off.**

- Emergency stop plate depressed.
- Main fuse blown.

R • Switch on main switch.

- Release emergency stop plate.
- Check main fuse and replace if necessary.

The main fuse is located next to the main switch.

S • Turbine produces excessive noise when running**C • Turbine rotor defective****R • Replace turbine rotor**

Observe instructions for turbine.

S • No fiber optics light function**C • Fiber optics not preselected**

- High-pressure lamp defective

R • Preselect fiber optics (section A 6.1)

- Replace high-pressure lamp

Observe separate OMA instructions included with instruments and motors.

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S • Instruments cannot be operated with spray**C** • Main cock in surgery shut off

- Spray not preselected
- Spray adjustment ring on instrument closed

R • Open main cock

- Preselect spray via unit foot control
- Open spray adjustment ring

S • Spray function on instruments unsatisfactory**C** • Spray nozzles clogged or lime deposits**R** • Unscrew spray cover and pierce nozzles with a needle**S** • Instruments leak**C** • O-rings on motor and Multiflex coupling defective**R** • Replace O-rings**S** • Water in return air filter**C** • O-rings on Multiflex coupling damaged**R** • Replace all O-rings on Multiflex coupling**S** • Suction hoses do not work**C** • Slide in conical section closed

- Strainers in suction connector clogged
- Vacuum exhauster does not work
- Sensor in amalgam collector or basket filter in collector soiled

R • Open slide

- Clean strainers
- Check vacuum exhauster fuse
- Clean sensor or filter (see separate OMA instructions for amalgam separator)

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S • Polymerization light does not work

C • Halogen lamp defective

R • Replace halogen lamp (page A 7-6)

Do not touch the halogen lamp reflector.

S • Results of polymerization unsatisfactory

C • Light emitting surface on light conductor soiled

- Light conductor damaged
- Lamp is old and used up

R • Clean light emitting surface

- Replace light conductor
- Replace lamp

S • Dental scaler tip does not oscillate

C • Tip not tightened

- Tip broken

R • Hand-tighten tip with supplied changing tool

- Replace tip

S • SONICflex handpiece leaks

C • Coupling not tightened

- O-rings on Multiflex coupling defective

R • Tighten union nut on hose

- Replace O-rings (see OMA instructions of Multiflex coupling)

S • Dental chair does not move

C • Safety cutout activated (display on control panel)

R • Check safety cutouts and eliminate cause of shutoff (section A 4.1)

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A 10 Optional accessories**A 10.1 3-function handpiece**

A second 3-function handpiece can be installed on the assistant's unit.

A 10.2 Polymerization light Poly LUX

A polymerization light can be installed on the assistant's unit.

A 10.3 Hydrocolloid kit

A hydrocolloid kit can be installed on the assistant's unit.

A 10.4 Saliva ejector

A vacuum-operated or a water-operated saliva ejector can be installed on the assistant's unit.

Criteria for the vacuum-operated saliva ejector:

- An amalgam separator Multi System Type 1 is present.
- Water pressure insufficient (below 3 bar).

Criteria for the water-operated saliva ejector:

- Low-cost water supply and adequate pressure (above 3 bar).
- Only low power vacuum exhauster present in office.
- Backflush is possible.

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A 10.5 Automatic separator with rinse unit

The automatic separator can be installed in the spittoon.

It is required if a vacuum-operated saliva ejector is used and there is no external separator.

The automatic separator separates the vacuum (vacuum exhauster) from the secretions.

A 10.6 X-ray viewers

A KaVo X-ray viewer for 5 x 5 cm X-ray films can be installed on the 1060 S/SK and 1060 T/TK.

If cervical table 1561 is present, an X-ray film viewer suitable for all film sizes (1440) can be installed on the 1060 G.

A 10.7 Dental scalers**A 10.7.1 Sonosoft LUX dental scaler**

A Sonosoft LUX kit can be installed in the unit.

A 10.7.2 SONICflex 2000 N / 2000 L dental scaler

A SONICflex 2000 N or 2000 L dental scaler may be operated with the unit.

A 10.8 KAVOLUX 1410

All 1060 units can be equipped with a unit or ceiling-mounted KAVOLUX operating light.

A 10.9 Cervical table 1561

Cervical table 1561 can be installed in conjunction with a mounting adapter (1060 G/GK only).

A 10.10 Right-hand armrest

The Comfort and Progress versions of the dental chair can be equipped with a right-hand armrest.

This armrest can be moved out of the way with one hand to allow the patient to get on or off the chair.

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A 10.11 Water boiler

The boiler is used to heat the tumbler filler water, instrument spray and 3-function handpiece.

It is installed in the supply unit.

A 10.12 Water disinfection system

The water disinfection system operates as a continuous disinfection system, i.e. it monitors water consumption and adds disinfectant in proportion to the amount of water consumed.

All water consumers with the exception of the hydrocolloid, bowl rinse and water jet pump are supplied with degerminated water in this way.

The water disinfection system can be easily installed in the supply unit.

A 10.13 Amalgam separator Multi System Type 1 (kit 1690 A)

Prescribed by law in Germany, Switzerland and Sweden.

This amalgam separator can be installed externally on all 1060 units.

A 10.14 Chair foot control

The optional chair foot control allows AP1, LP and all manual chair functions to be controlled by foot.

A 10.15 Water pressure accumulator 1320

All 1060 units can be equipped with a water pressure accumulator.

When the water quality is bad or water pressure is low, the instruments and the 3-function handpiece are supplied from the accumulator.

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The KaVo dental scaler is used to scale natural teeth and to remove superfluous cement at an oscillation frequency of 30 kHz. The teeth are easily, gently and painlessly scaled using light, brush-like movements along the surface of the tooth.

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The intensity is electronically controlled via a foot switch to leave the dentist's hands free for treatment.

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Water is supplied through the working tip, and the water amount may be regulated by adjusting the water control cock according to the tip used. The amount of water supplied must be such that the instrument tips atomize the water with the intensity of their oscillation. The optimum setting is reached when the water spray is at about body temperature.

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Three instrument inserts are supplied as standard accessories.

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For reasons of simplified handling and hygiene, the instrument inserts should be stored in the changing tool.

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If the tips have been mechanically damaged (i.e. if they have been dropped, bent, etc.), it is likely that they will not perform properly.

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Inserting the instrument insert

The instrument tip contained in the changing tool is manually screwed into the handpiece by turning it clockwise. Then tighten. The changing tool must move easily on the thread. If there is resistance during the first few revolutions, loosen the instrument and repeat the procedure.

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The changing tool serves the following purposes:

1. as a key to change the instrument inserts
2. to store the instrument inserts
3. to sterilize the instrument inserts
4. to protect the instrument inserts screwed into the handpiece.

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Sterilization

The instrument inserts in the changing tool may be sterilized in an autoclave (135° C).

Without changing tool, the instrument inserts may also be sterilized in a hot-air sterilizer.

Safety Directions:

The materials and contours of the inserts are developed and checked in long series of experiments.

However it can happen that in consequence of the repetition of stress or damage (dropping the instrument on the ground or mechanical changing of the original form) a breach can arise.

Around that it is necessary to check the instrument inserts before every treatment. We recommend to renew every 9-12 month the inserts if they are used frequently.

For additional safety we recommend to test the instrument inserts with a mechanical load of approx. 10 N (see sketch). 10 Newton = 1 kg

If the instrument, the cable or the electrical connections are damaged it is not allowed to continue the work. All required repairs have to be done by authorized technicians.

Application notes

Removing tartar and bacterial plaque from natural teeth is unessential for perfect oral hygiene and thorough periodontal treatment. Scaling the crowns and roots of teeth is the basis of any dental treatment. A reliable diagnosis is hardly possible without clean teeth. Contrary to the difficulties encountered when manually removing hard plaque (instrument guidance, pure scratching, risk of injury to gingiva and lips, etc.), intra-oral ultrasound scaling may be applied gently, quickly and simply. Particularly in difficult areas (necesses, narrow passages, molar areas), ultrasound scaling is a highly efficient means with great prophylactic effect which supports dental therapy. This refined and thorough technology will also encourage routine dental scaling on a larger number of patients on often overloaded dental practices.

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No booklet of operating instructions can give complete details concerning the various indications for the use of the scaler or recommendations for the manner of treatment to be followed. This must be left experienced and reputed authorities.

The book published by "Die Quintessenz" in Berlin can be considered an excellent guide with regard to optimum scaling techniques. This book drew the attention to the ultrasound method at a rather early stage. It is important that the dentist (but also the "dental hygienist" or assistant with a similar job description) be convinced of the necessity of careful treatment.

Removal of the total hard and soft dental plaque provides for the type of dental hygiene which counteracts progressive inflammation of the gingiva and, in fact, of the entire periodontium.

Plaque formation is an important factor in the etiology of gingival diseases. The bacteriological element is of extreme importance in this context. Rough, septic and irritating dental plaque is very frequently the cause of severe inflammations and the loss of individual teeth or entire groups of teeth, predominantly in the highly susceptible mandible area. Teeth which are "spared" by the patient because of pain or because the corresponding teeth in the maxilla, or mandible, respectively, are missing, are another object of dental plaque which will lead to inflammation. Due to the sensitivity of the gingiva surrounding these teeth, the gentle ultrasound treatment is to be considered beneficial for the dentist as well as the patient. The removal of irritating dental plaque, particularly tartar, is therefore an inevitable prerequisite for successful treatment in most cases.

The consequences of dental plaque should already be prevented on young patients by informing and treating them. A preventive and supportive measure is counseling on targeted and through oral hygiene.

The instrument and its handpiece are gripped firmly like a pen. When they are gripped correctly, the instrument tip may be easily guided and rolled. Supporting the instrument on the adjacent tooth will improve the scaling method and prevent the instrument from bouncing or sliding off. The instrument must be guided back and forth with light, gentle and smooth movements. Try it out on extracted teeth whose plaque-covered areas have been impregnated with methylene blue.

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Cleaning the crowns

If the surface of the tooth is impregnated with (methylene blue, fuchsin or with special preparations), it is possible to find out exactly where plaque is to be removed and where it has been completely removed.

Various types of instrument inserts permit optimum application even in places with difficult access.

The recommended method is to apply the instrument tip vertically and in parallel to the tooth. The instrument tip should therefore be moved in parallel to the surface of the tooth, but never with the edge in order to prevent damaging the substance of the teeth through nicks. Do not press the tip on too hard in order to treat teeth and the instrument gently.

After the scaling procedure, we recommend polishing the surface of the tooth using the KaVo prophylactic head No. 31 GR with the appropriate rubber inserts and a fine paste in order to achieve an improved preventive caries treatment.

Removing subgingival plaque

Removing subgingival plaque is a difficult task. Working "blind" requires secure and gentle instrument guidance without pressure. Blowing on the gingival margin using well-tempered warm air may provide a brief view in some areas. Carefully check for subgingival plaque using a probe prior to beginning the scaling procedure.

The instrument tip must be inserted all the way to the bottom of the pocket while lifting off the pocket wall as little as possible. All surfaces may be reached with the appropriate straight or angular instrument inserts. The supplied water will carry the removed plaque off. In case of sensitivity, particularly in the subgingival area, the use of surface anesthetics or local anesthesia is recommended. As for narrow passages, the combined usage of ultrasound and hand instruments is the method of choice if even the last remaining concretions are to be removed using one's sense of touch. This combined method has been repeatedly recommended by scientists. Responsible manufacturers of ultrasound equipment will note this in their operating instructions.

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Rubbing in pieces of dental plaque already removed with a hand instrument must be avoided. This may happen if the hand instrument has not been cleaned after insertion in the pocket and removal of plaque. Ultrasound instruments do not cause this problem due to their oscillation and the supplied water.

Note that the water supply may not be excessive. A "mild" jet of water is desirable for scaling subgingival plaque. As already mentioned, we recommend applying the instrument at the bottom of the periodontal pocket and gently moving it backwards.

The ultrasound instrument may not be moved back and forth in a swinging movement as usual with hand instruments.

If the instrument edge is applied in parallel to the surface of the tooth, the handpiece will point at the vestibule of the mouth. The working end may then lie flat against the tooth (root) surface without stretching the soft tissue during the scaling operation.

The curettage

The curettage of periodontal pockets which is considered a periodontal-surgical measure is possible with an ultrasound instrument. In case of a periodontal pocket, removal of the dental plaque may not cause healing, so that the actual healing process must be initiated by additionally removing the diseased epithelial layer.

Once this has been removed with the ultrasound instrument under local anesthesia, primary healing will occur. Bleeding and secretion will stop.

In the curettage, the active side of the instrument insert, whose sides are blunt, is applied towards the gums and gently moved in a lateral direction at the highest frequency setting. The "peeling" effect is achieved by moving the instrument out in a shovel-like movement. The supplied water should have a shower-like effect in order to completely remove the pieces of tissue which have been peeled off during the ultrasound application.

This treatment is buccal, lingual and approximal.

The inside wall should be smooth, which may be checked using the instrument tip (without turning the instrument on).

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The gingiva should be pressed on as much as possible. The patient should refrain from eating for at least two hours and should not chew on the side which has received the treatment.

Rinsing with warm water is recommended for the first day following the operation. The patient should not brush his teeth for three days.

The place of operation must be kept clean using cotton swabs which have been soaked in water.

Formation of plaque must absolutely be prevented, if necessary by follow-up treatment in the practice.

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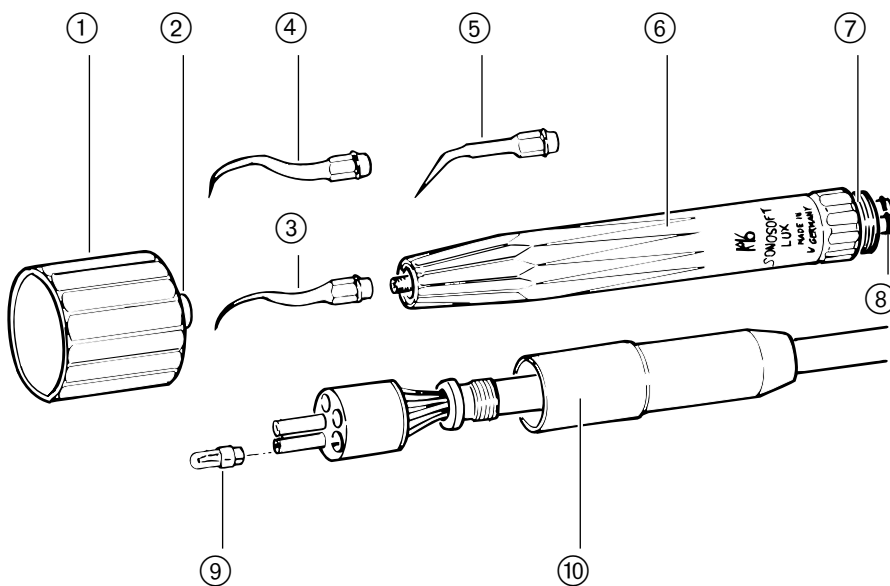
TECHNICAL DATA AND REQUIREMENTS

Drilling template No.	486 7403 / 486 7404	Drainage rate	
Installation plan No.	486 7405 Bl. 2/3/4	without jet pump	5 l/min
		with jet pump	8 l/min
		Gradient of water drain pipe	from unit at least 10 mm per meter
Electric supply cable	3 x 1.5 mm ²		
Free end above floor	1000 mm		
Input voltage	110/110/120 V AC	Air connection	R 1/2
	130/220/230/240 V AC	above floor	40 mm
Frequency	50/60 Hz		
Factory-set input voltage	See name plate	Air inlet pressure	5.2 - 7 bar
		Air inlet flow rate	80 l (S.T.P.)/min
Max. power consumption		Suction connection	Ø 40 mm
(incl. KAVOLUX 1410)		above floor	20 mm
at 100 - 130 V	570 VA		
at 220 - 240 V	925 VA	Suction vacuum rate / vacuum flow	
Fuse in building	automatic circuit breaker C16		≥ 65 mbar / ≥ 500 NI/min at inlet into unit
			≥ 19 mbar / ≥ 280 NI/min at suction nozzle
Potential equilibration line	See VDE 0107		static < 180 mbar.
Above floor	1000 mm		Values applicable for KaVo measuring set, Order No. 411 8500
Heat output	72 - 3000 (72 - 2830) kJ/h ¹⁾		
Approval symbols	FTZ-Nr. 411/88 VDE/GS	Weight	gross / net - / approx. 175 kg
Water supply	According to local waterboard regulations. DIN 1988 as well as twin 6 (issue 01/94) are in force. A safety device type BA (three chamber system) is to be installed before each dental unit.	¹⁾ Depending on operating mode	
		²⁾ In the event of high water hardness (> 12 dH), a water softening unit must be installed in the ion exchange process with disinfection system. An excessively low water hardness may promote algal growth.	
Water quality	Drinking water	Floor characteristics: The quality of the floor finish must correspond to the load carrying capacity for buildings according to DIN 1055 sheet 3; compressive strength according to DIN 18 560, part 1.	
Water hardness			
Total hardness:	1.50 - 2.14 mmol/l ²⁾		
	△ 8,4 - 12 dH		
pH-value	7,2 - 7,8		
Water supply filtration	80 µm		
Water connection	R 1/2		
above floor	40 mm		
Areas with brackish stagnant water should be excluded.			
Water inlet pressure	2.5 - 6.0 bar		
Water inlet flow rate			
without jet pump	5 l/min		
with jet pump	8 l/min		
Drain connection	Ø 40 mm		
above floor	20 mm		

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KaVo Dental GmbH, D-88400 Biberach/Riss